Kryptopyrrol (L 120)

Testanleitung

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Test Instructions

Sampling test kit includes:

- 1 x Request Form*
- 1 x Test Instructions*
- 1 x Outer Tube light protected*
- 1 x Urine Monovette*

Instructions:

The Sarstedt-Urine-Monovette must be filled with **morning urine**. For an accurate result it is important to use the entire morning urine, i.e. the bladder should not have been emptied in the last six hours before. Please keep in mind not to take any cink or vitamine B6 for 7 days before taking the test as it may influence your results.

- After getting up, catch the **first morning urine** in the urine collection cup.(or in an appropriately clean container).
- Fill the Sarstedt-Urine-Monovette (to do this, unscrew the small cap, put on the yellow plastic tip, pull on the piston until the tube is completely filled and then break off the piston at the predetermined breaking point, remove the plastic tip and unscrew the cap again).
- Plastic cup and pipette can then be disposed.
- Please label the enclosed label and stick it onto Sarstedt-Urine-Monovette.

Please label sample tube:

- *Patient name*
- *Date of sampling*
- *Material -> check the box "urine"*

Please put the labelled urine tube into the transport tube (protective containers for postal dispatch) and put these into the shipping envelope together with the completed request form.

Important: Send the test set by post as quickly as possible. Shipment over the weekend and public holidays should be avoided.