Gludorphin und Casomorphin (C710)

Testanleitung

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Gludorphin and Casomorphin (C710)

Test Instructions

The test set contains the following materials:

1 x request form

1 x instructions

1 x pipette for transferring the urine into the urine tube (white lid)

1 x urine tube with a white lid (contains a stabilizer = white substance)

1 x outer container (brown)

1 x sticker for labelling the urine tube

*An urine collection cup can be ordered at biovis' by fax with the order form shipping material **Art.-Nr. 63***

Instructions:

- The urine tube has to be filled with the **first morning urine**.
- Collect the first morning urine in the urine collection cup (or in a suitable clean vessel) after getting up.
- Tap lightly the bottom of the urine tube (white lid) several times onto a solid surface, so that the (non-poisonous) stabilizing powder accumulates at the bottom.
- Fill the urine tube (white lid) with the enclosed pipette.
(Please make sure that the stabilizing substance remains in the tube)
- Now place the urine tube in the brown outer container.
- Then the urine collection cup and pipette and can be discarded.

Please label sample tube:

- **Patient name**
- **Date of sampling**
- **Material -> check the box "urine"**

Please put the labelled urine tube into the black transport tube (protective containers for postal dispatch) and put these into the shipping envelope together with the completed request form.

Important: Send the test set by post as quickly as possible.

Shipment over the weekend and public holidays should be avoided.