## Pee-Spot (A560)

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## Pee-Spot (A560)

**CAUTION!**  
Test is not suitable  
for diabetics


### Test Instructions

#### The test set contains the following materials:

- 1 x request form
- 1 x instructions
- 1 x vessel with sugar
- 1 x urine tube (Pee-Spot)
- 1 x 2-liters-urine collection container
- 1 x tube with stabilizing fluid (chlorhexidine - please store cool and dark)
- 1 x outer container
- 1 x name label

#### Instructions:

The test has to be carried out in the morning on an empty stomach. Please dissolve the sugar in the vessel in 200 ml of lukewarm water and drink the solution on an empty stomach. Fill the stabilizing fluid in the urine collection container and collect the urine for about 4 to 5 hours. During this time, no sugary foods (e. g. bread, potatoes, rice, pasta, cake) and drinks should be consumed. After three hours, you can eat a light meal without sugar (e. g. fish, meat, raw food, vegetables). If a sufficient amount of urine is collected, open the tight screw cap (1) of the Pee-Spot-tube (2) and pull out the Pee-Spot on the small handle (3); Keep the tube and the lid.

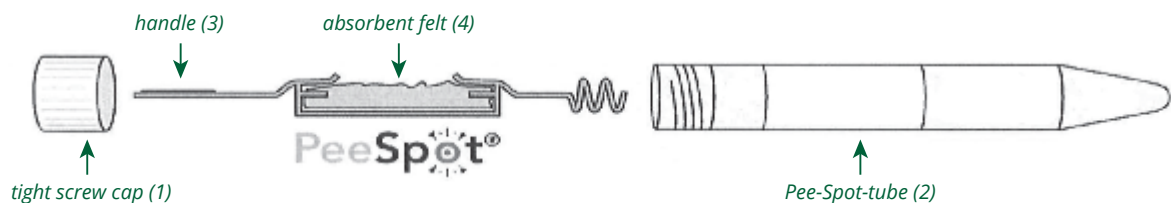Now hold the Pee-Spot by the small handle (3) and dip the absorbent felt (4) in the urine collection container. Then put the handle with the soaked felt back in the Pee-Spot-tube and close tight with the screw cap.

#### Please label sample tube:

- **Patient name**
- **Date of sampling**
- **Material**

Please put the labelled urine tube into the black transport tube (protective containers for postal dispatch) and put these into the shipping envelope together with the completed request form.

**Important:** Send the test set by post as quickly as possible.  
Shipment over the weekend and public holidays should be avoided.