

Schwermetalle vor und nach Chelattest (I113 , I114 , I130)



Testanleitung

Im Testset sind folgende Materialien enthalten:

- 1 x Anforderungsbogen**
- 1 x Testanleitung**
- 1 x 2 Liter Urinsammelbehälter**
- 4 x Transportumröhrchen**
- 4 x Urinmonovette**
- 1 x Urinbecher**

Zur Testvorbereitung sollte ca 2 Wochen vor Chelatierung das Sicherheitsprofil Chelat (I 927) durchgeführt werden:

Blutuntersuchung: 2SZ, NaF, Hep, EDTA

Anleitung Chelatierung:

- 2 Urinmonovetten mit Urin befüllen (vor Chelat – falls gewünscht)
- Chelatierung
- Urinprobe nach Chelat sammeln: **3-5 h nach Beginn der Chelatierung**
Je höher die Glomeruläre Filtrationsrate (GFR) desto kürzer reicht die Sammelzeit.
Aus dem Sammelurinbehälter die weiteren 2 Urinmonovetten befüllen.

Bitte Probengefäße unbedingt beschriften mit:

- **Namen**
- **Datum**
- **Material**
- **vor Chelat**
- **nach Chelat**

Bitte geben Sie die beschrifteten Urinmonovetten in die weißen Transportumröhrchen (Schutzgefäße für den Postversand) und diese zusammen mit dem ausgefüllten Anforderungsbogen in die Postversandtasche.



Wichtiger Hinweis: Bringen Sie das Testset möglichst schnell auf den Postweg.
Ein Versand über das Wochenende und über Feiertage hinweg sollte vermieden werden.

Heavy Metals before and after Chelate Test (I113 , I114 , I130)



Test Instructions

The test set contains the following materials:

- 1 x Request Form**
- 1 x Test Instructions**
- 1 x 2 Liter Urine Collection Container**
- 4 x Transport Tube**
- 4 x Urine Monovette**
- 1 x Urine Cup**

For test preparation, the Safety Profile Chelate (I-927) should be made approx. 2 weeks before chelating.

Blood examination: 2SZ, NaF, Hep, EDTA

Test Instruction - Chelating:

- 2 Fill urine monovettes with urine (before chelating – if desired)
- Chelating
- Collect urine sample after chelating: **3-5 h after start of chelating**.
The higher the Glomerular Filtration Rate (GFR), the shorter the collection time will be.
Fill the other 2 urine monovettes from the urine collection container.

Bitte ProbengefäÙe unbedingt beschriften mit:

- **Name**
- **Date**
- **Material**
- **before chelating**
- **after chelating**

Now put the labelled saliva specimen tubes into the transport tube (white) and put it into the shipping envelope together with completed request form. Ship as soon as possible!



Important: Send the test set by post as quickly as possible.

Shipment over the weekend and public holidays should be avoided.