

Stuhlprobengewinnung



Testanleitung

Im Testset sind folgende Materialien enthalten:



1 x Anforderungsbogen

1 x Testanleitung

2 x Stuhlfänger

2 x Stuhlröhrchen

2 x Transportumröhrchen

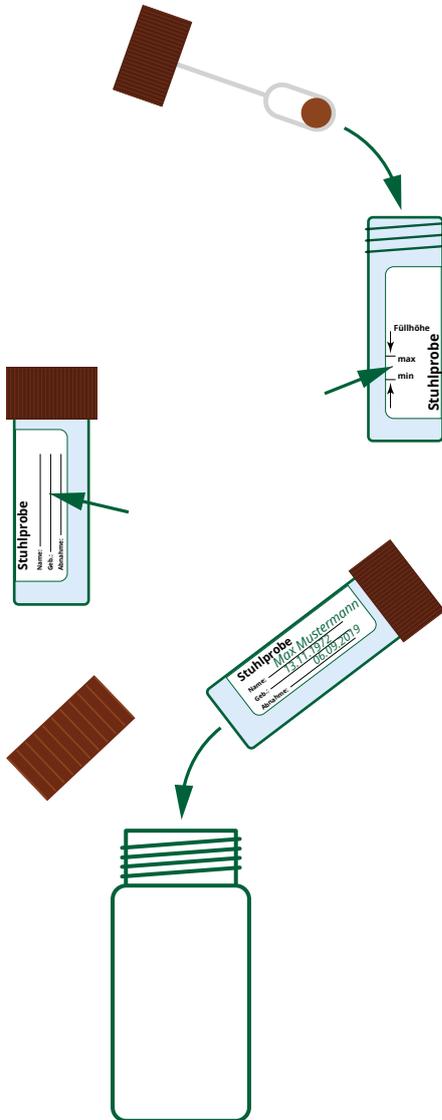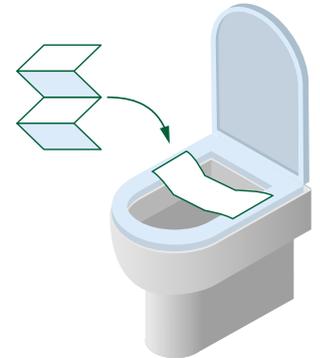

Um eine hohe diagnostische Sicherheit der Stuhluntersuchung zu gewährleisten, beachten Sie bitte folgende Punkte bei der Probenentnahme:

1. Bitte verwenden Sie zur Stuhlgewinnung den **beiliegenden Stuhlfänger**. Der Stuhl sollte nicht mit dem Toilettenbecken oder Wasser in Berührung kommen.
2. Bitte schrauben Sie das **Stuhlröhrchen (brauner Deckel)** auf und entnehmen Sie mit dem Löffel an mehreren Stellen Stuhlproben. Bitte befüllen Sie das Stuhlröhrchen bis zur Markierung.
3. Schrauben Sie den Deckel mit dem Löffel wieder auf das Stuhlröhrchen. Bitte beschriften Sie das Etikett mit dem Namen, Geburtsdatum und dem Abnahmedatum und stecken es in das beiliegende **Transportumröhrchen (weiß)**.
4. Bitte packen Sie das Transportumröhrchen und den ausgefüllten unterschriebenen **Anforderungsbogen** in die Postversandtasche und kleben diese zu.
5. Bitte geben Sie die Postversandtasche bevorzugt in einer Postfiliale ab.

Wichtiger Hinweis: Bitte nicht am Freitag (Wochenende) oder vor Feiertagen versenden!!!

Taking stool samples



Test Instructions

The test set contains the following materials:



1 x request form

1 x instruction manual

2 x stool catcher

2 x stool tubes

2 x transport tubes

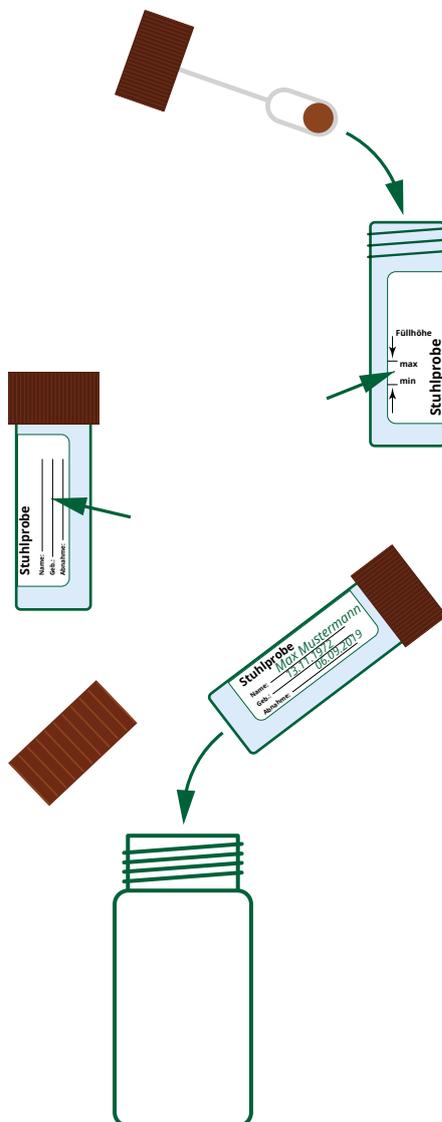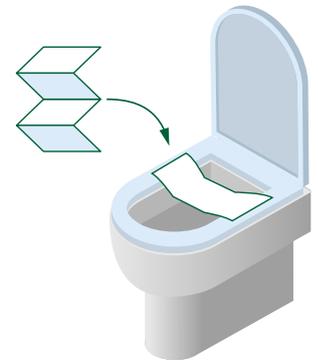

To ensure high diagnostic standard of the stool test please consider the following when taking the stool sample:

1. To avoid getting the stool in contact with water of toilet bowl, please use the **enclosed stool collector**.
2. Screw the lid **with the spoon** open to take samples at several spots and fill the stool tubes. Fill the tube to the line.
3. Then screw the lid with the spoon back on the tube. Please put your name and date on each tube. Wrap the tubes in the fleece cloth and put them in the sealbag. Seal the **sealbag** by removing the silver strip.
4. Please put the sealbag and **the signed request form** in the shipping box.
5. Please bring the shipping box to the post office and put on postage.
Please note: Please send only on Monday, Tuesday or Wednesday! Do not send before the weekend and before public holidays!