

## Östrogen-Metaboliten (F325)



### Testanleitung

#### Im Testset sind folgende Materialien enthalten:

- 1 x Anforderungsbogen**
- 1 x Testanleitung**
- 1 x Zentrifugenröhrchen stabilisiert**
- 1 x Transferpipette**
- 1 x Transportumröhrchen**
- 1 x Namensetikett**

#### Anleitung:

Das Urinröhrchen mit weißer Kappe muss mit dem zweiten Morgenurin bis zur Markierung oben gefüllt werden.

- Fangen Sie den zweiten Morgenurin nach dem Aufstehen in dem Urinauffangbecher auf (oder in einem entsprechend sauberen Gefäß).
- Befüllen Sie mit der beiliegenden Pipette das Urinröhrchen mit der weißen Kappe bis zur Markierung oben - ca. 5 mm unter dem geschlossenen Deckel. Bitte achten Sie darauf, dass die Stabilisatorsubstanz in dem Röhrchen verbleibt.
- Plastikbecher und Pipette können anschließend entsorgt werden.

#### Bitte Probengefäße unbedingt beschriften mit:

- **Namen**
- **Datum**
- **Material = Urin ankreuzen**

Bitte geben Sie die beschrifteten Urinmonovetten in die weißen Transportumröhrchen (Schutzgefäße für den Postversand) und diese zusammen mit dem ausgefüllten Anforderungsbogen in die Postversandtasche.



**Wichtiger Hinweis:** Bringen Sie das Testset möglichst schnell auf den Postweg.

Ein Versand über das Wochenende und über Feiertage hinweg sollte vermieden werden.

## Estrogen Metabolites (F325)



### Test Instructions

#### The test set contains the following materials:

**1 x Request Form**

**1 x Test Instructions**

**1 x Centrifuge Tube stabilized**

**1 x Transfer Pipette**

**1 x Transport Tube**

**1 x Name Label**

#### Test Instruction - Chelating:

The urine tube with white cap must be filled up to the mark above with the second morning urine.

- After getting up, catch the second morning urine in the urine collection cup.(or in an appropriately clean container).
- Use the enclosed pipette to fill the urine tube with the white cap up to the mark above - approx. 5 mm below the closed lid. Please ensure that the stabilizer substance remains in the tube.
- Plastic cup and Pipette can then be disposed.

#### Please be sure to label sample containers with:

- **Name**
- **Date**
- **Material = tick urine**

Now put the labelled saliva specimen tubes into the transport tube (white) and put it into the shipping envelope together with completed request form. Ship as soon as possible!



**Important:** Send the test set by post as quickly as possible.

Shipment over the weekend and public holidays should be avoided.