



Taking stool samples



Instruction manual

Stool sampling set:

- 1 x request form
- 1 x instruction manual
- 2 x stool tubes
- 1 x stool collector
- 1 x sealbag
- 1 x shipping box

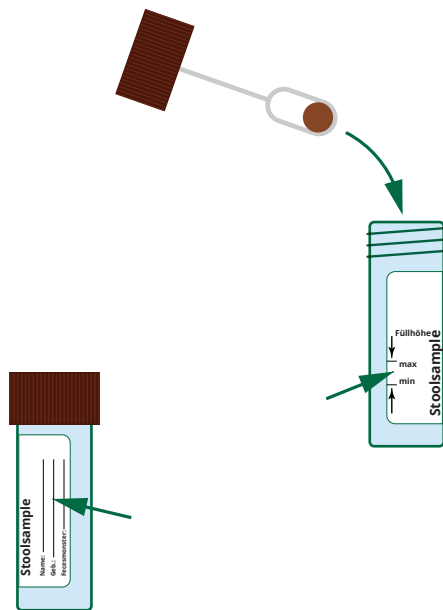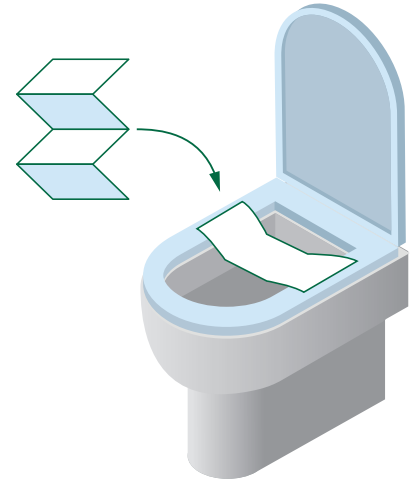

To ensure high diagnostic standard of the stool test please consider the following when taking the stool sample:

1. To avoid getting the stool in contact with water of toilet bowl, please use the **enclosed stool collector**.
2. Screw the lid **with the spoon** open to take samples at several spots and fill the stool tubes. Fill the tube to the line.
3. Then screw the lid with the spoon back on the tube. Please put your name and date on each tube. Wrap the tubes in the fleece cloth and put them in the **sealbag**. Seal the sealbag by removing the silver strip.
4. Please put the sealbag and **the signed request form** in the shipping box.
5. Please bring the shipping box to the post office and put on postage.

Please note: Please send only on Monday, Tuesday or Wednesday! Do not send before the weekend and before public holidays!



Afname fecesmonster



Handleiding

De testset bevat de volgende onderdelen:

- 1 x aanvraagformulier
- 1 x handleiding
- 2 x fecesbuisjes
- 1 x papieren fecesvanger
- 1 x lekvrije sealbag
- 1 x verzendoos

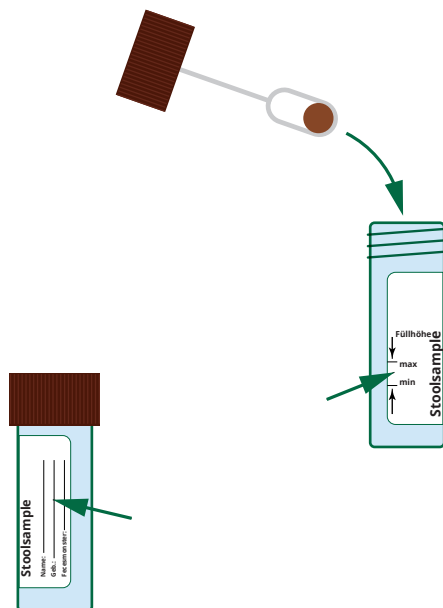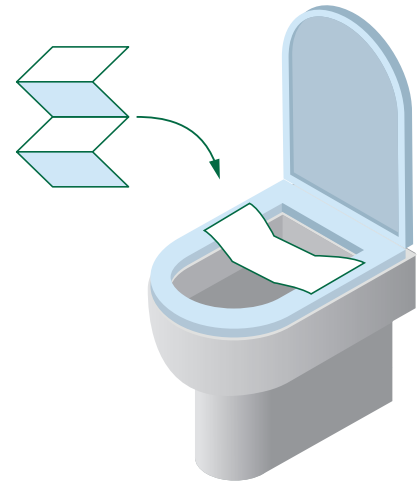

Voor nauwkeurige en sluitende laboratorium diagnostiek graag de volgende punten in acht nemen:

1. Gebruik de **bijgevoegde fecesvanger** voor de afname van het fecesmonster zodat de feces niet in aanraking komt met water en/of toiletpot.
2. Schroef het **buisje** (bruine deksel) open en neem met de geïntegreerde lepel op verschillende plekken feces af. Vul het busje **tot de streep**.
3. Schroef het deksel inclusief lepel weer op het fecesbuisje. Graag naam en datum feces afname op ieder busje vermelden. Wikkel de busjes in het vliesdoek en doe alle busjes in de sealbag. Plak de **sealbag** dicht door de zilveren strip te verwijderen.
4. Doe de sealbag en het ingevulde en ondertekende **aanvraagformulier** in de **verzendoos**.
5. Breng de doos envelop naar een postkantoor of brievenbus.
Let op: Graag uitsluitend versturen op maandag, dinsdag of woensdag. Niet versturen vóór het weekend en voor feestdagen!